


How Does A DDC Dolphin Bedpan Washer Disinfector Work?


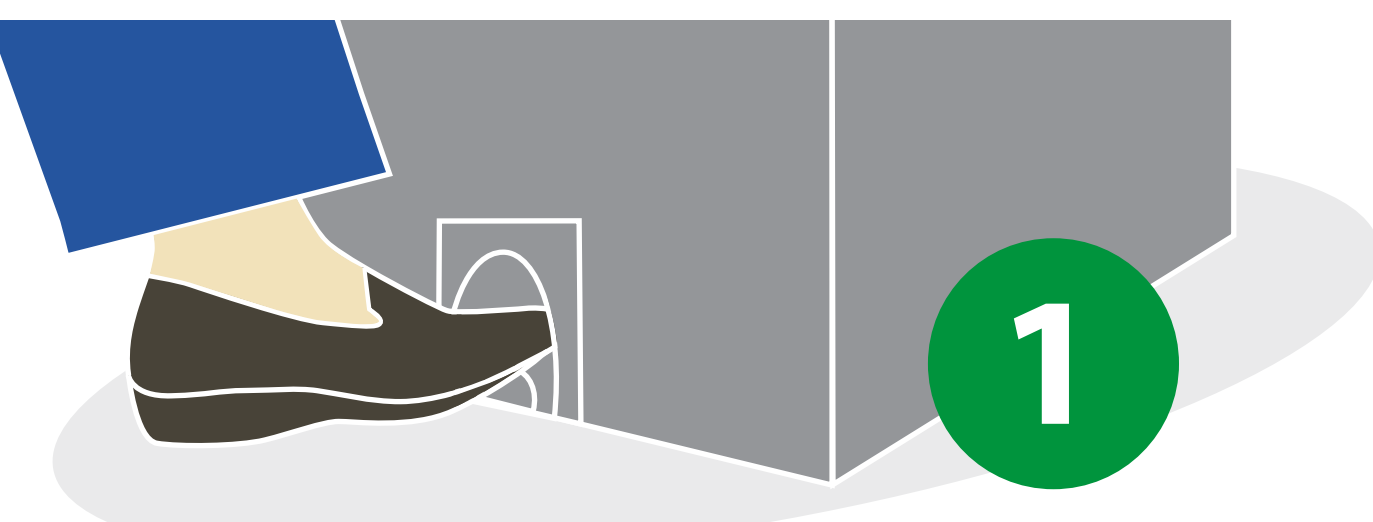


1

Place your foot in the foot cup/tap the foot plate to open the machine. Optional programmable audio (available on top-loading machines) will give you safe loading and operation instructions.

2

Load the used bedpans, urine bottles or commode pots as specified by your machine instructions to ensure thorough disinfection. Human waste, toilet paper and flushable wipes are the only items which are safe to dispose of in a bedpan washer.

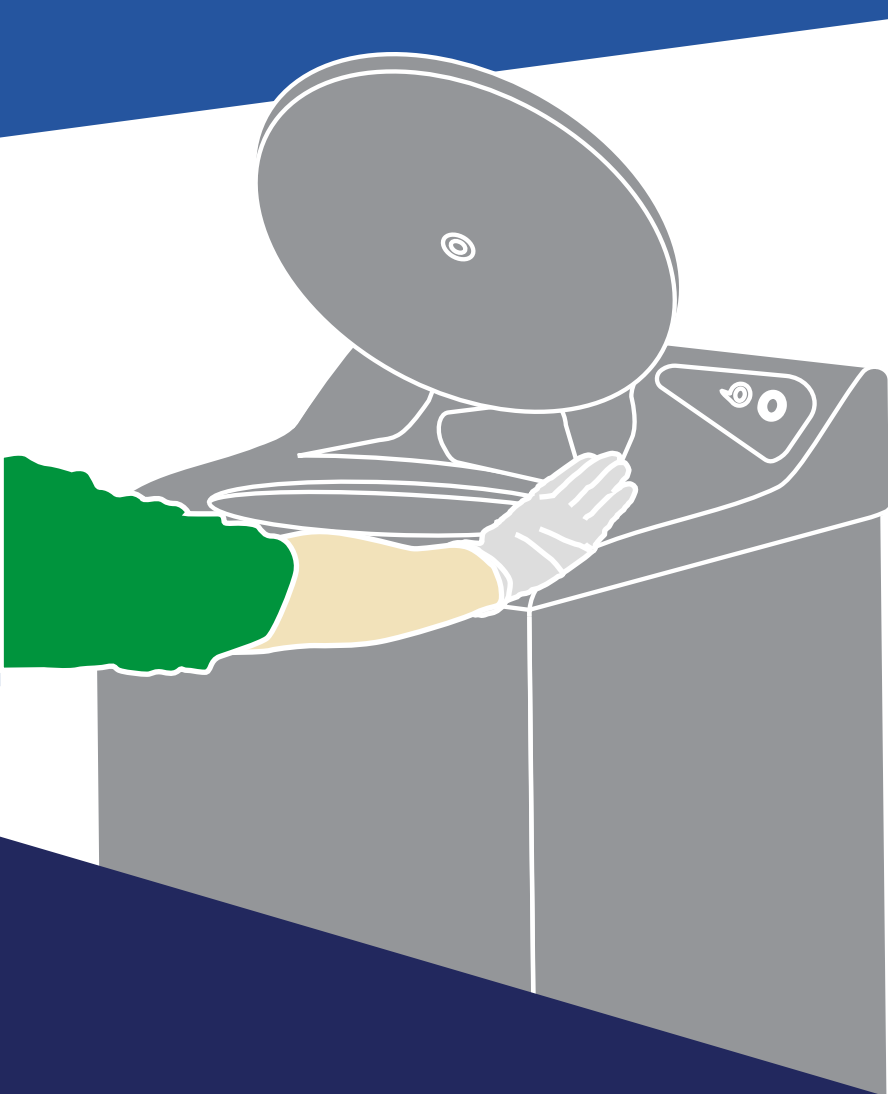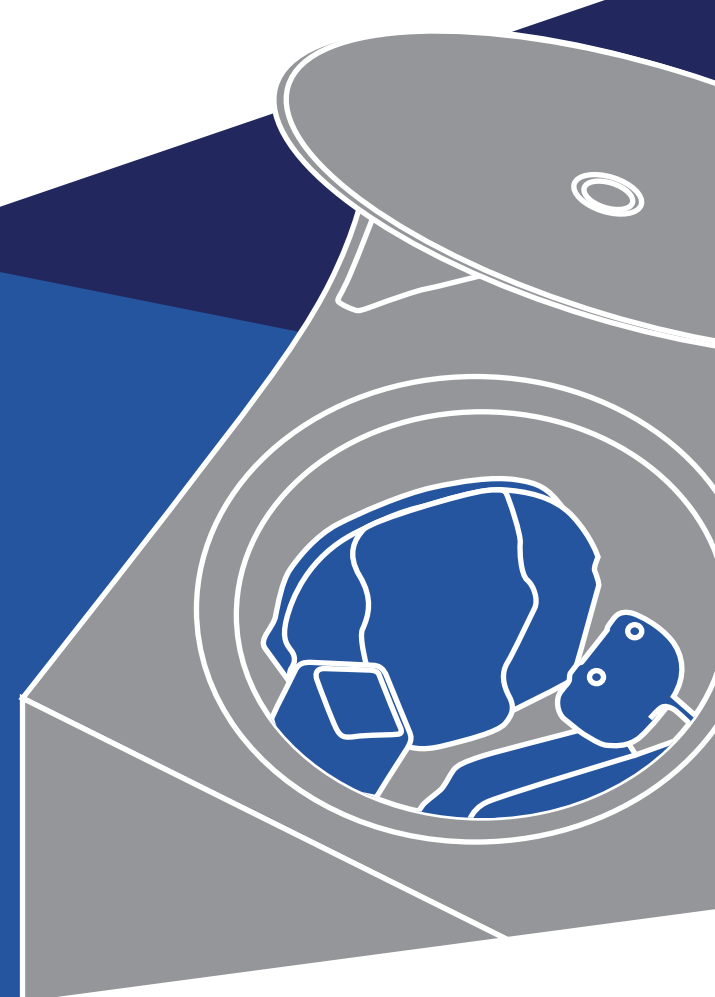

3

Depending on the model and setup of your machine:

- The lid will close and the cycle will start automatically after a pre-set time.
or
- Hold your hand over the infrared sensor to close the lid and start the cycle.
or
- Close the lid firmly and press the green button to start the cycle.

4

Strategically placed fixed and rotating wash nozzles direct water towards the bedpans to maximise dispersal of waste. The cycle can take as little as 6 minutes from start to finish.





5

Steam will gradually heat the central drum up to the 80°C required to kill bacteria, so you can be confident that the cleaning process is effective.


6

A steady flow of water to the drain removes waste and dirty water, ensuring drains are kept blockage free.

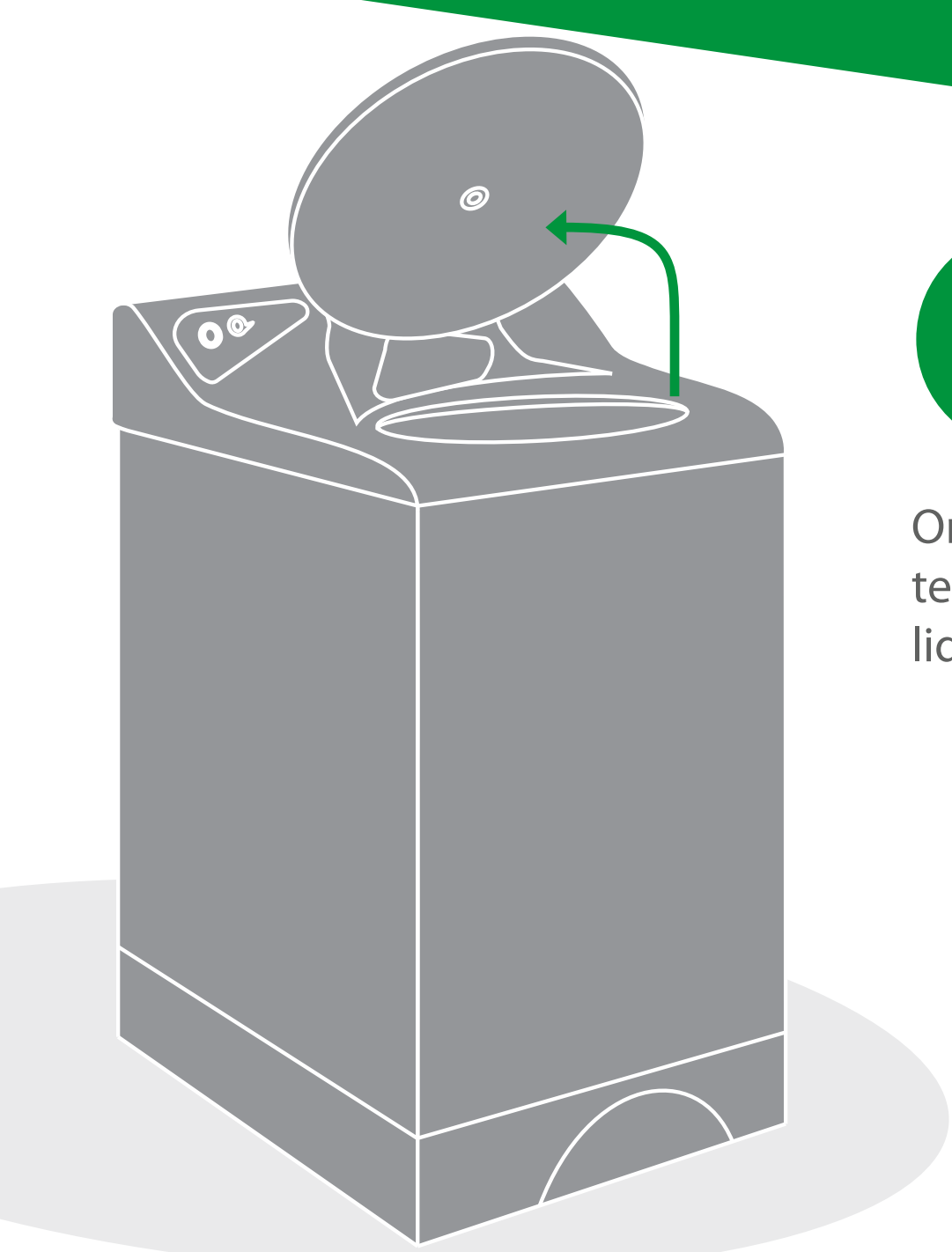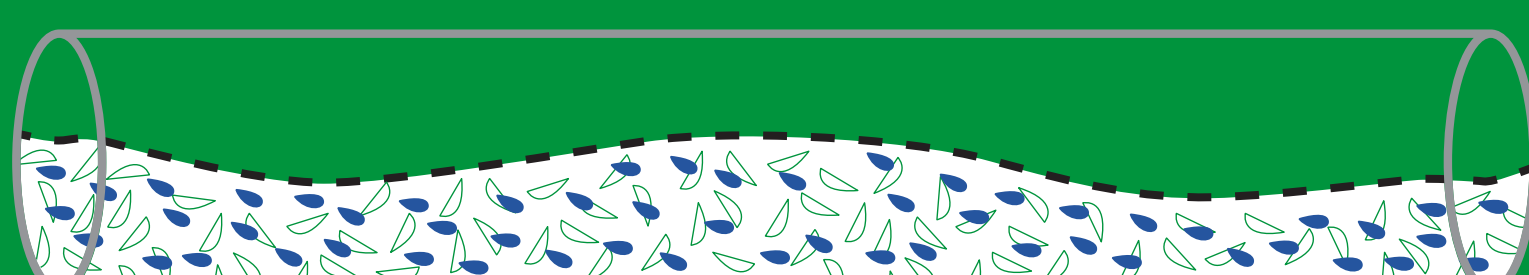

7

Once the drum has cooled to a safe temperature of 60°C, the machine lid or door can be opened.

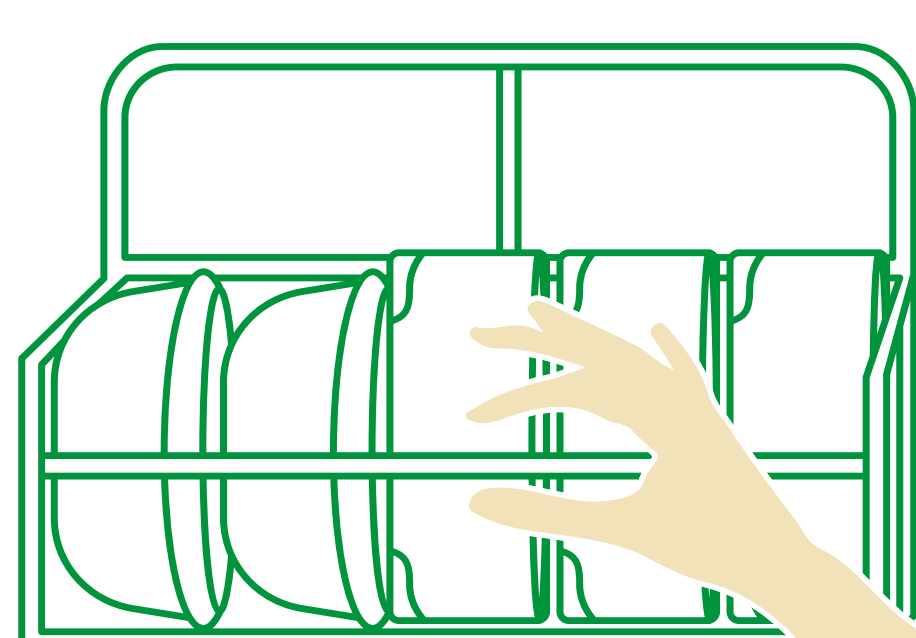

8

To unload the machine, place your foot in the foot cup/tap the foot plate to open the machine, remove the clean utensils and place on a storage rack to dry.

Do you need a bedpan washer for your facility?
Get in touch today to discuss your requirements and learn more about the DDC Dolphin range

